



UltraFill

Provider quick start guide





PHILIPS





RESPIRONICS

sense and simplicity

Startup checklist


To facilitate proper setup and filling, follow the steps listed below. Before attempting to troubleshoot a problem, always check to make sure that the system has been set up correctly and that the filling process was initiated properly.


	Action	How to verify
1	 <p>Tubing is properly connected between UltraFill and the oxygen concentrator. Check for kinks.</p>	Inspect visually.
2	 <p>Oxygen concentrator is powered on.</p>	Green light on top panel should be lit. If not, check that the power cord is plugged in and the power button is depressed. If the problem persists, contact your service provider.
3	 <p>Oxygen concentration level is properly set. EverFlo should be at or below 3 LPM.</p>	Check the concentrator's flow meter.
4	 <p>UltraFill is powered on.</p>	Green light on top panel should be lit. If not, check that the power cord is plugged in and the power button is depressed. If the problem persists, consult the Troubleshooting section in this guide.

		Action	How to verify
5		<p>Conservor or regulator is in the “0” position.</p>	<p>Inspect visually.</p>
6		<ol style="list-style-type: none"> 1. Raise the cylinder rest to fill a 1.0L cylinder, or 2. Lower the cylinder rest to fill a 2.0L cylinder. 	<p>Inspect visually.</p>
7		<p>Connect the cylinder to the UltraFill device.</p>	<p>One or more lights on the fill-level indicator is solid.</p>
8		<p>Depress start/stop button. Filling has started.</p>	<p>If cylinder is not full, one light on the fill-level indicator will flash. If full, all 4 LEDs will be lit solid.</p>
9		<p>Filling complete. Release cylinder from UltraFill.</p>	<p>All lights on the fill-level indicator are solidly lit.</p>
10		<ol style="list-style-type: none"> 1. Set conservor or regulator to your prescribed setting. 2. Make sure the shut-off knob is completely opened (Turn counterclockwise). <p>Cylinder is ready for use.</p>	<p>Inspect visually.</p>

Troubleshooting

Once you have verified that the setup and filling procedures have been followed correctly, troubleshoot the problem using the following tables.

LED and audible condition		
		
● Green:	Continuous	
● Yellow:	Off	
● Red:	Off	
Audible alarm:	Silent	
Condition		
The device is turned on and working properly. Oxygen pressure and purity from the concentrator are good.		
Troubleshooting step		
NA		
Corrective actions		
NA		

LED and audible condition		
		
● Green:	Continuous	
● Yellow:	Off	
● Red:	Off	
Audible alarm:	Silent	
Condition		
The device is not filling.		
Troubleshooting step		
<ul style="list-style-type: none"> • Verify that the cylinder is connected to the fill coupler (fill level LEDs lit). • Verify that the fill Start/Stop button is depressed (either a fill level LED will be blinking or all 4 LEDs will be constantly lit). 		
Corrective actions		
<ul style="list-style-type: none"> • If all 4 LEDs are constantly lit, the cylinder is full and ready for use. • If all 4 fill level indicator LEDs are not lit and none are blinking, again depress the fill Start/Stop button. If this does not start the filling process, please contact product support. 		

LED and audible condition



● Green:	Continuous
● Yellow:	Flashing
● Red:	Off
Audible alarm:	Periodic

Condition

The device has detected a low or high pressure fault.

Troubleshooting step

- Check oxygen flow at the transfer tubing connection of concentrator.
- Verify flow out of the EverFlo is greater than 10 liters per minute.
- Check for kinks in the oxygen feed tube between the concentrator and UltraFill.
- Ensure the OCD or regulator is set to the “0” position.

Corrective actions

- If oxygen flow is less than 10 liters per minute, contact product support.
- If the line is kinked, straighten the tubing and retry filling.
- Set in “0” position and retry filling.
- If oxygen pressure is correct, tubing is not kinked, and the OCD/regulator is properly set, contact product support.

LED and audible condition



● Green:	Continuous
● Yellow:	Continuous
● Red:	Off
Audible alarm:	Off

Condition

The device has detected a possible oxygen purity fault (UltraFill purity is less than 90 percent, cylinder purity is greater than 90 percent.)

Troubleshooting step

Check the concentrator’s oxygen purity at the patient cannula port.

Corrective actions

- If concentrator purity is less than 90 percent, remove the concentrator’s inlet filter and retry filling.
- If warning subsides, replace the filter. If the warning persists, contact product support.
- If the concentrator’s oxygen purity is greater than or equal to 90 percent, contact product support.

LED and audible condition



● Green:	Continuous
● Yellow:	Continuous
● Red:	Flashing
Audible alarm:	Constant

Condition

The device has detected a possible oxygen purity fault (purity is less than 90 percent) and the oxygen within the cylinder is less than 90 percent.

Troubleshooting step

- Empty oxygen from the cylinder.
- Check the concentrator's oxygen purity at the cannula port.

Corrective actions

- If concentrator purity is less than 90 percent, remove the concentrator's inlet filter and retry filling.
- If warning subsides, replace the filter. If the warning persists, contact product support.
- If the concentrator's oxygen purity is greater than or equal to 90 percent, contact product support.

LED and audible condition



● Green:	Off
● Yellow:	Off
● Red:	Continuous
Audible alarm:	Constant

Condition

The device has detected an internal failure.

Troubleshooting step

- Contact product support.

Corrective actions

- Contact product support.

Safe transport

Reasonable care should be taken when moving the UltraFill device across an uneven surface. Follow the general guidelines below when attempting to move UltraFill across the floor, through doorways, and into an elevator.



Correct transport method over uneven surface

The proper method for moving the UltraFill device is to first remove the cylinder, then either lift the unit using its two handles or pull the device along its long orientation using one handle.



Incorrect transport method over uneven surface

Never attempt to move the UltraFill device with the cylinder attached or by pushing the top of the unit along its horizontal axis.

**Philips Healthcare is part
of Royal Philips Electronics**

How to reach us

www.philips.com/healthcare
healthcare@philips.com

Asia

+49 7031 463 2254

Europe, Middle East, Africa

+49 7031 463 2254

Latin America

+55 11 2125 0744

North America

+1 425 487 7000

800 285 5585

(toll free, US only)

Philips Respironics
1010 Murry Ridge Lane
Murrysville, PA 15668

Customer Service

+1 724 387 4000

800 345 6443

(toll free, US only)

Philips Respironics

International Headquarters

+33 1 47 28 30 82

Respironics, EverFlo, Millennium, and UltraFill are trademarks of Respironics, Inc. and its affiliates. All rights reserved.

Please visit www.philips.com/respironics



© 2011 Koninklijke Philips Electronics N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

Broudy AE 3/18/11 MCI 4104080 PN 1083679